EN COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution
Brilliant cleansing for long-lasting comfort

Important – Please read carefully and keep this package insert for future reference.

COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution cleans, rinses, disinfects, stores, removes surface deposits including lipids and proteins, and re-wets soft (hydrophilic) contact lenses.

Caring for your contact lenses with COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution.

Always wash and dry your hands thoroughly before handling your soft contact lenses.

Cleaning:

- Place a few drops of COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution on each lens surface.
- Rub each side gently from the center to the edge for 2-4 seconds. Never rub in a circular motion, it may tear the lens and not clean the outer edge surface.

Rinsing:

- Rinse each side of the lens for at least 5 seconds with COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution.
- Place lenses in the correct chambers of the AMO lens case (blue for left - white for right).
- Fill both chambers with fresh COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution. Replace lens case caps and soak for at least 6 hours for disinfection, protein and lipid removal. Your lenses are now ready for wear. Rinse the lens case with COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution and leave to air dry.

Storage: You may leave your lenses in the unopened lens case in COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution for up to 30 days continuously. After 30 days, cleaning and disinfection must be repeated before wearing your lenses.

Intensive Cleaning: COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution removes protein from your lenses during cleaning and disinfection. This will help to keep your lenses smooth for enhanced wearing comfort. Where the level of protein deposits is unusually high a more intensive protein removal may be necessary. Your eye care practitioner will advise if you require protein removal tablets.

WARNINGS:

All instructions must be followed. Failure to do so may result in serious eye infection or injury. If eye irritation occurs immediately discontinue lens wear and contact your eye care practitioner.

Precautions:

General: Use COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution with soft contact lenses, including silicone hydrogel lenses
Do not use if allergic to any of the ingredients
Do not wear your lenses longer than prescribed
Discard any lenses that are damaged or dehydrated
Avoid using aerosols when wearing contact lenses
Attend regular check-ups with your eye care practitioner
Keep out of the reach of children

Use: Always use COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution with the AMO COMPLETE lens case provided
Do not use if tamper-evident seal is missing or broken
Always use fresh solution
Always manually rub your lenses with solution for at least 2-4 seconds before soaking
Do not swim, shower or bath while wearing contact lenses
Do not use saliva or put lenses in your mouth to rewet
Do not use after the expiry date
To avoid contamination do not allow the bottle tip to touch any surface and avoid contact with eyes
Do not swallow the solution
Do not rinse and store your lenses or lens case with tap water

Storage: Close the bottle tightly immediately after use
Store between 5°C to 25°C

Discard and Expiry: Use before the expiry date on label and carton
Discard any remaining solution 90 days after first opening the bottle
Discard any remaining solution in your contact lens case before disinfecting your lenses again
Always discard your old lens case and use the new one provided

Product: COMPLETE RevitaLens® Multi-Purpose Disinfecting Solution

Ingredients: Aloxidine 0.00016%; Polyquaternium-1 0.0003%; Boric acid; Sodium borate decahydrate; Tetronic 904; Edetate disodium (EDTA); Trisodium citrate dihydrate; Sodium chloride and Purified water.

This product does not contain phthalates. This product does not contain BPA.

AMO Ireland
Block B, Liffey Valley Office Campus
Dunyrallye, Co Dublin, Ireland

AMO, COMPLETE and COMPLETE RevitaLens are trademarks owned by or licensed to Abbott Laboratories, its subsidiaries or affiliates. TETRONIC is a trademark of BASF Corporation

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© 2015 Abbott Medical Optics Inc.